




In all versions of Dexis, open any patient and select the **Dexis** logo in the upper right-hand corner.

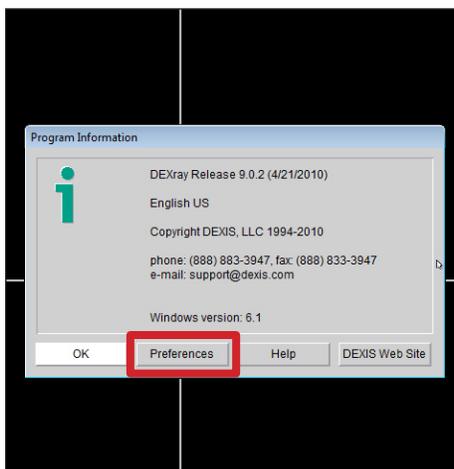


In all versions of Dexis, select **Preferences** from the Program Information window that appears.


Due to variations between different versions of Dexis, the remainder of this document has been divided into two sections.

If you are using Dexis version 7, 8 or 9, proceed to the next row to configure your software.

If you are using Dexis Imaging Suite (Dexis 10), skip to the bottom row of this document for instructions on how to configure your software.

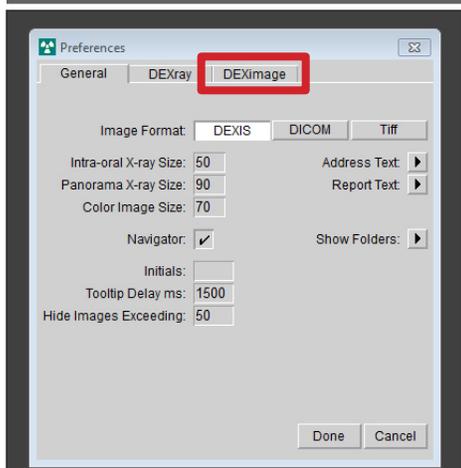

Select the **DEXImage** tab from the Preferences window that appears.

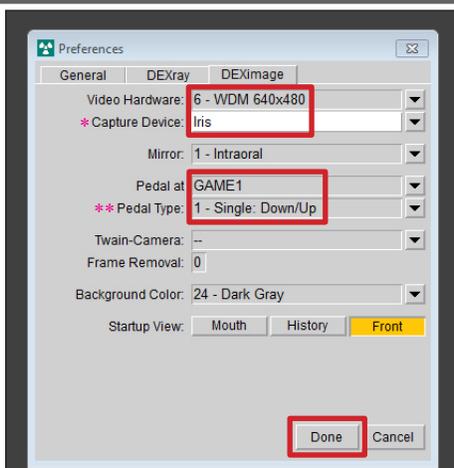


Set your **Preferences** so that they match those shown in the illustration.
 * Version 9 only. ** Versions 8 and 9 only.


Set **DEXImage Preferences** as follows:

- Set **Video Hardware: WDM 640x480**
- Set **Capture Device: Iris***
- Set **Pedal at: GAME1**
- Set **Pedal Type: Single: Down/Up****
- Select **Done**.

*This can be set to Automatic initially. **Iris** will not populate until after you have launched the Live Video window once after camera install.
 This **Pedal Type install utilizes quick presses to toggle between live video and freeze, and a long press (hold) to save the frozen image. To capture every image (freeze, save, live), you can set **Pedal Type: Double: Capture/Move**.

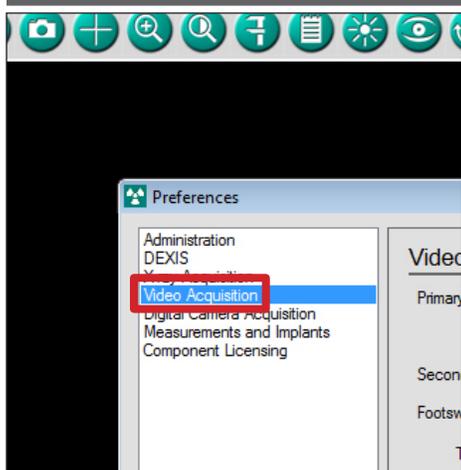

Select the **Video Acquisition** option from the **Preferences** window that appears.

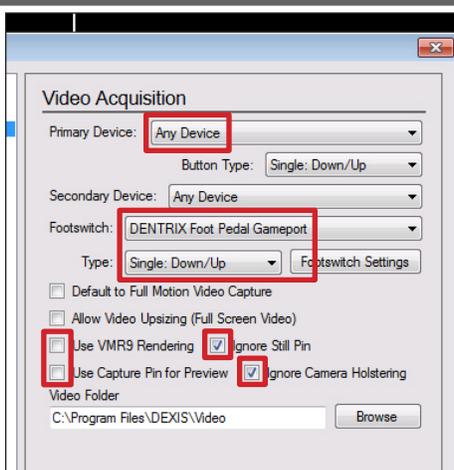

Set your **Preferences** so that they match those shown in the illustration.

Set **Video Acquisition Preferences** as follows:

- Dropdown Lists:**
- Primary Device: Any Device***
- Footswitch: DENTRIX Foot Pedal Gameport**
- Type: Single: Down/Up****
- Checkboxes:**
- Uncheck: **Use VMR9 Rendering**
- Check: **Ignore Still Pin**
- Uncheck: **Use Capture Pin for Preview**
- Check: **Ignore Camera Holstering**
- Select **OK**.

You **MUST** restart your computer at this time.

*If you added **Digital Doc Camera** through Extra Hardware Installs when installing Dexis Imaging Suite, select that from the **Primary Device** list.
 This **Pedal Type install utilizes quick presses to toggle between live video and freeze, and a long press (hold) to save the frozen image. To capture every image (freeze, save, live), you can set **Pedal Type: Double: Capture/Move**.