


CDR DICOM 5.4 Client Installation Instructions

Please proceed to the following steps to install CDR DICOM for Windows 5.4 Server.

Browse the CD and open the Client folder. Right click on **CDR DICOM for Windows Setup.exe** (Application) and choose **Run as Administrator**.

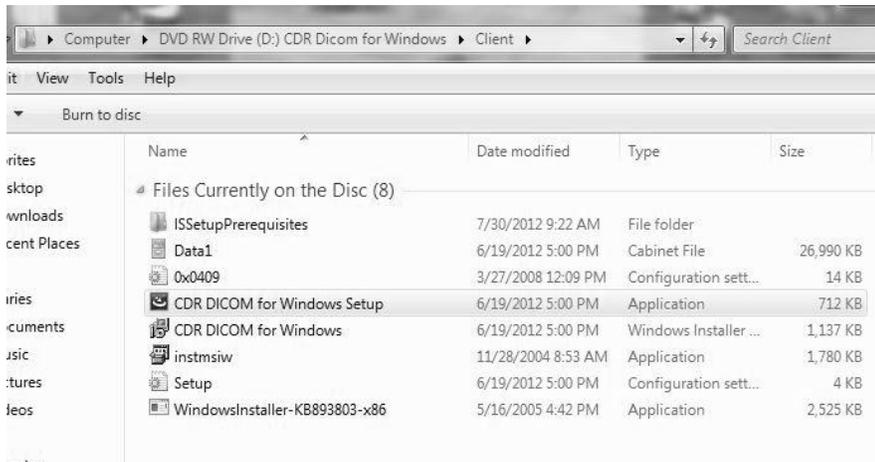

It will commence to preparing to install.



Choose **Next**.



Fill in the circle for **I accept the terms in the license agreement** and choose **Next**.




Enter the User Name and Organization you wish to use. Choose **Next**.



Leave the circle filled in for **Complete** and choose **Next**.



Choose **Install**.

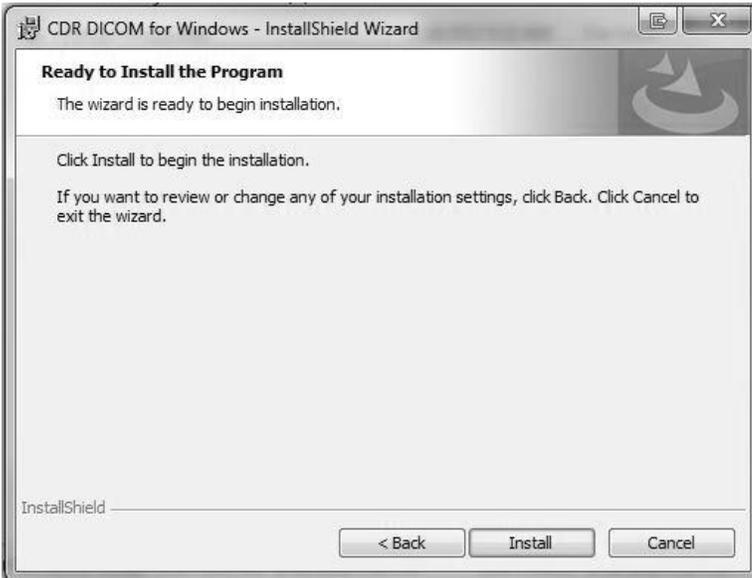


Installation progress will commence.



Choose **Finish**.



At this time, you should now be able to open CDR DICOM for Windows. The first time it is opened, it will prompt for connection. Choose the correct Server Name from the drop down list in the top box. Leave the box filled in to **Connect with my Windows login credentials** and click **OK**. It should now take you to the software.

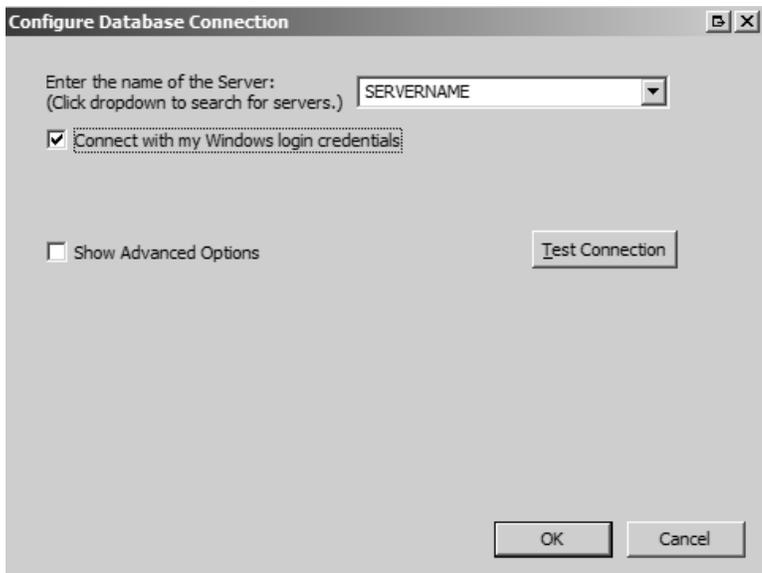

For any questions please contact the Patterson Technology Center at 800.475.5036 and ask to speak with Digital support.